





FLEXIBLE CONTACT LAYER WITH TLC-NOSF

Indication: chronic wounds located in cavity or awkward areas

- Accelerates wound healing
- Re-balances the metabolic environment in chronic wounds
- Ideal for awkward locations, cavity and deep wounds
- Painless and atraumatic removal



Supplied in boxes of individually pouched and sterile dressings, ready to use

Sizes	5 x 7 cm
	10 x 10 cm
	15 x 20 cm












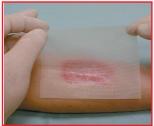





DESCRIPTION

■ **UrgoStart Contact** is a non-adhesive, non-occlusive lipidocolloid matrix withTLC-NOSF, an innovativeTechnology developed by the Laboratoires Urgo.

COMPOSITION: UrgoStart Contact is made of a polyester mesh

impregnated with a formulation of hydrocolloid particles (Carboxymethylcellulose), petroleum jelly and NOSF (Nano-Oligo Saccharide Factor).

METHOD OF STERILIZATION: Sterilized by ionizing radiation.

PROPERTIES

- In contact with wound exudate, the TLC combined with NOSF gives the matrix specific properties, triggering the optimum healing process.
 - •Thanks to its hydrocolloid particles, the TLC forms a lipidocolloid gel, creating a moist environement that promotes the healing process, so that key cells involded in the repair process (fibroblasts, keratinocytes, macrophages) can exert their action.
 - NOSF provides properties in addition to those of the TLC: in contact with wound exudate, NOSF forms a gel which binds preferentially to damaged areas to interact with the wound micro-environment by limiting the harmful action of MMPs (Matrix Metallo Proteases).
- The TLC-NOSF combination leads to the restoration of conditions promoting the granulation of chronic wounds, in which a metabolic imbalance due to an excess of MMPs has led to a continuous degradation of the extra-cellular matrix, and possibly to delayed healing.
- **UrgoStart Contact** is flexible and comformable; it helps ensure a good fit of the dressing to all wound shapes (leg ulcers, pressure ulcers, diabetic foot ulcers).
- UrgoStart Contact does not adhere either to the wound or the surrounding skin: the dressing changes are painless for the patient and atraumatic for the newly formed tissues in the wound.

INDICATIONS

- **UrgoStart Contact** is indicated for chronic wounds (leg ulcers, pressure ulcers, diabetic foot ulcers, long standing acute wounds).
- Flexible and very comformable, UrgoStart Contact is especially

suitable for lining cavity or awkwardly placed wounds and for wicking deep wounds.

DIRECTIONS FOR USE

METHOD OF USE

- Preparation of the wound:
 - Clean the wound with normal saline.
 - If an antiseptic is used first, rinse the wound thoroughly with saline solution before applying **UrgoStart Contact**.
- Application of the dressing:
 - Remove the protective wings from UrgoStart Contact.
 - Apply UrgoStart Contact to the wound.
 - **UrgoStart Contact** can be cut using sterile equipment to fit the dressing to the wound.
 - Cover **UrgoStart Contact** with a secondary dressing suitable to absorb the level of wound exudate and hold the secondary dressing in place with a comforming bandage, adhesive tape or an elasticated tubular bandage.
 - For exuding wounds, use **UrgoStart**, the soft adherent hydrocellular version of **UrgoStart Contact**.
- Dressing changes:
 - **UrgoStart Contact** dressing may be changed every 2 to 4 days, and left in place for up to 7 days depending on the level of exudate and the wound condition.
 - The recommended treatment duration for **UrgoStart Contact** is a minimum of 4 to 5 weeks.

PRECAUTIONS

- UrgoStart Contact adheres to latex surgical gloves. It is therefore recommended that gloves be moistened with saline in order to make the handling of UrgoStart Contact easier.
- If there are clinical signs of local infection and depending on clinical judgement, the wound can be treated with an antibacterial

dressing before applying UrgoStart Contact.

- In the absence of clinical data as a first line treatment in acute wounds, and in the treatment of Epidermolysis Bullosa (even for long standing lesions), it is not recommended to use **UrgoStart Contact**.
- In the case of an atypical ulcer presenting induration or overgranulation, **UrgoStart Contact** should only be used after checking the absence of wound-related malignancy in order not to delay the diagnosis.
- In the case of deep wounds and fistula wounds, a section of the **UrgoStart Contact** dressing should be left visible to enable easy removal.
- ■The product's action on retriggering the healing process may possibly cause stinging or painful sensations on commencement of treatment with **UrgoStart Contact**. This rarely warrants suspension of treatment.
- Do not re-sterilise the dressing.
- Store UrgoStart Contact dressings flat.
- Single use sterile individual packaging: re-using a single use dressing may lead to risks of infection.

CONTRAINDICATIONS

- In order not to delay any optimal treatment, UrgoStart Contact is contraindicated in wounds which are not likely to heal such as:
 - Cancerous wounds,
 - Fistula wounds which may reveal a deep abscess.

