

Instillaquill®


Instructions for use

Instillaquill® is a sterile single use extension tube to fit an Instillagel® syringe. It is designed to provide 2% lidocaine gel to the cervical canal prior to gynaecological investigation procedures

The use of Instillagel® to locally anaesthetise the cervix avoids the use of a hypodermic syringe and its inherent trauma.

Instillagel® also acts as a lubricant allowing smoother passage of instruments - and has an antibacterial action provided by 0.25% Chlorhexidine.


1. Open blister pack where indicated.

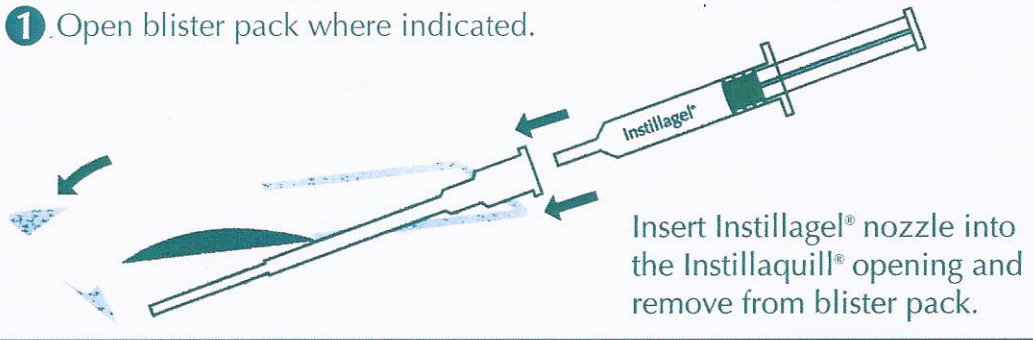


2. Open vagina by use of a speculum and insert Instillaquill® into the cervical canal.



3. Prime Instillaquill® with gel. Introduce Instillaquill® to a point where the collar on the tube reaches the neck of the cervix.

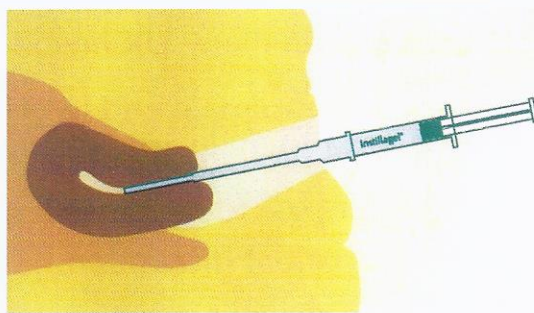

4. Slowly withdraw Instillaquill® whilst at the same time injecting sufficient gel at a steady rate to coat the cervical canal. On completion remove Instillaquill® and dispose of together with the used Instillagel® syringe.

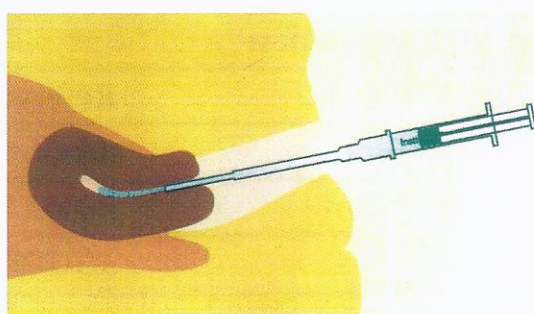

Note: It is recommended that following insertion of Instillagel®, 5 minutes be allowed to achieve optimum anaesthetic effect.

Instillagel® is available in 5ml and 10ml presentations.

5ml Instillagel® is indicated for use in femal gynaecological investigation. Instillaquill® is supplied sterile in a blister pack and should be discarded if the pack has been damaged.

sales@mdsanz.com

1 / 16-18 Tennyson Street,
Williamstown North
VIC 3016, Australia
Tel 1300 857 230
NZ 0800 994 951
www.mdsanz.com



MDSOLUTIONS
AUSTRALASIA

STERILE EO



CE
0088

Instillaquill®

Extension tube facilitating the use of Instillagel® anaesthetic gel in gynaecological investigations.

The use of Instillagel® in gynaecological investigations is now facilitated by the availability of Instillaquill® - a single use extension tube allowing the delivery of the gel to the cervix. The narrower distal part of Instillaquill® makes it suitable for use in both multiparous and nulliparous women, whilst the adjacent collar acts as a guide to the length of tube inserted into the cervical canal. Instillaquill® is indicated for use in day case surgery or minor out-patient procedures such as endometrial biopsy or out-patient hysteroscopy.

Instillagel®

Instillagel's anaesthetic, antiseptic and lubricating actions coupled with its unique syringe delivery system have made it the gel of choice in many UK hospitals.

Widely used prior to male and female catheterisation Instillagel® has also allowed urological out-patient procedures - such as flexible cystoscopy - to be carried out with maximum efficiency and minimum patient discomfort.

Product Information

Sterile / Single use only

Product	Reference	pack quantity
Instillagel® 5ml	343343	10 syringes
Instillagel® 10ml	343367	10 syringes
Instillaquill®	40-200	10 quills

Prescribing Information:

Composition:

Each 100g gel contains:	
Lidocaine Hydrochloride	2.0g
Chlorhexidine Digluconate solution	0.25g
Methyl Hydroxybenzoate	0.06g
Propyl Hydroxybenzoate	0.025g

Uses:

Catheterisation, cystoscopy, Exploratory and intra-operative investigations, exchange of fistula catheters, protection against iatrogenic damage to the rectum and colon.

Gynaecological investigation.

Dosage and administration:

Unless otherwise prescribed by doctor:

Urethral catheterisation: instil 5-10ml of gel into the urethra. The anaesthetic effect begins after 3-5 minutes.

Contra-indications, Warnings, Precautions and Interactions:

Instillagel® must not be used in patients with known hypersensitivity to the active ingredients (amide-type anaesthetics, chlorhexidine and alkyl hydroxybenzoates) or any of the excipients. It should not be used in patients who have damaged or bleeding mucous membranes. Use with caution in patients with impaired cardiac conditions, hepatic insufficiency and in epileptics. Difficulty in swallowing may occur with an increased risk of aspiration and biting trauma. Use with caution in patients receiving antiarrhythmic drugs.

Undesirable effects:

In spite of the proven wide safety range of Instillagel®, undesirable effects of lidocaine are possible in severe injury to the mucosa and absorption may occur; for example, anaphylaxis, fall in blood pressure, bradycardia or convulsions.

Presentations:

Pre-filled disposable syringes; for single use only. 5ml and 10ml; packs of 10.

sales@mdsanz.com

1 / 16-18 Tennyson Street,
Williamstown North
VIC 3016, Australia

Tel: 1300 857 230

NZ: 0800 994 951

www.mdsanz.com



MDSOLUTIONS

A U S T R A L A S I A