

# DAYS OVER BED POLE

This Overbed Pole can be used in conjunction with a conventional or hospital bed. It is lightweight, sturdy and easy to assemble.

## Parts

1. Molded composite adjustable handle on webbing strap
2. Base frame
3. Middle section
4. Top section



## Specifications:

- Overall height: 1680mm
- Mold handle height adjustment: 1090– 1320mm
- Footprint: 760mm x 850mm
- Maximum User Weight: 135kg
- Net weight: 13kg

Check all parts for shipping damage. In case of shipping damage, do not use. Contact dealer/carrier for further instructions

## Warning

- Do not install or use this equipment without first reading and understanding this instruction sheet. If you are not able to understand warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment. Otherwise injury or damage may occur.
- Days over bed pole is not designed to support total body weight of an individual. Use it for assistance only.
- Always be aware of the location of boom tube, base assemblies and handle otherwise injury or damage may occur.
- Do not install or adjust the over bed pole while bed is occupied.
- After installation and before use, ensure the tube sections are fully engaged at all times.
- Secure strap between stoppers at top section as shown.
- After adjusting strap to suitable length, ensure strap is securely locked in place.
- Always locate the over bed pole so handle is directly above the individual using the handle. Handle should be grasped with both hands and pulled straight down. Do not pull handle at an angle otherwise injury or damage may occur.



## Help and advice

- It may be advisable to consult a physiotherapist or occupational therapist before choosing and using this equipment.
- Lifting poles may not be appropriate for people in certain circumstances, for example if they have poor trunk control or painful shoulders.
- If freestanding, care should be taken to ensure that the base is positioned properly under the bed, otherwise the frame may tip over. Furthermore, ensure that there is enough clearance for the base to slide under a divan bed where required.
- If you also use a mobile hoist for transfers, ensure that the lifting pole base does not prevent the host from being correctly positioned under the bed and that the overhead arm does not obstruct the lifting boom of the hoist.

Manufactured for P ..... V- ..... Tel: 1300 473 422 7 .....  
Unit 3, Bassalt Road .....  
Pemelwuy NSW 2145 AUSTRALIA .....  
Made in China

**PERFORMANCE**  
**HEALTH**

# Assembly Instructions (1-4)


## Double loop, strap, handle



(1) Pass strap through smaller loop on double-loop



(3) Route strap around bracket as shown

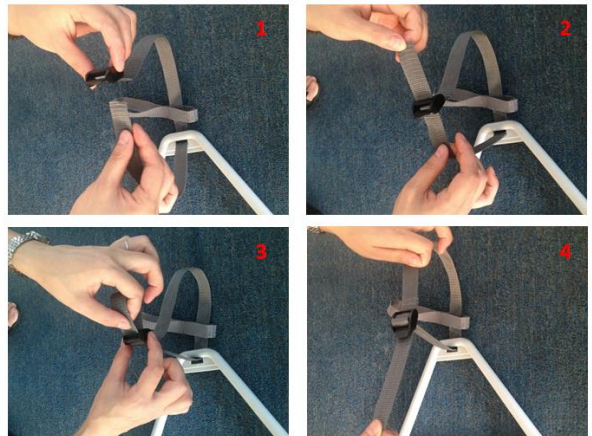

(2) Pass strap through slot on handle



(4) Put bigger loop of double loop between stoppers

