

Use and Maintenance of the Hollister Extended Wear Urinary Leg Bag



For Indwelling Catheter Drainage

1. If connecting to a Night Drainage Bag, cut a short length of the extension tubing and place on Leg Bag outlet.
2. Connect the remaining tubing to the Leg Bag inlet.
3. Connect tubing to the indwelling catheter.
4. Attach Leg Bag Straps to accommodate either horizontal or vertical placement on thigh or calf.
5. For night drainage attach the Night Drainage Bag to end of Leg Bag outlet.
6. Loosen or undo the Leg Bag Straps for sleeping, or wear in the horizontal position.
7. **Never** disconnect Leg Bag from catheter until due for change. This maintains a 'closed' system.
8. Replace the Leg bag with replacement of the catheter. (Or earlier at the discretion of your medical supervisor)
9. Further advice is available from the National Continence Helpline on 1800 330 066.

Product Ordering Codes - Ext. Wear Range
9348 – 540 ml Leg Bag with extension tubing and Leg Bag Straps.
9349 – 900 ml Leg Bag with extension tubing and Leg Bag Straps.

For Sheath Drainage

1. If connecting to a Night Drainage Bag, cut a short length of the extension tubing and place on Leg Bag outlet.
2. Connect the remaining tubing to the Leg Bag inlet.
3. Connect extension tubing to penile sheath.
4. Attach Leg Bag Straps to accommodate either horizontal or vertical placement on thigh or calf.
5. For night drainage attach the Night Drainage Bag to end of Leg Bag outlet.
6. Loosen or undo the Leg Bag Straps for sleeping, or wear in the horizontal position.
7. Remove Leg Bag and extension tubing daily to clean and dry.
8. Replace Leg Bag at sign of wear and tear. Extended wear times are often reported.



Hollister Customer Service
Toll Free:
Australia 1800 335 911
New Zealand 0800 678 669

Cleaning the Extended Wear Leg Bag

1. Empty all urine from Leg Bag into toilet.
2. Close outlet and fill Leg Bag with cold tap water – shake vigorously.
3. Empty water through drainage outlet into toilet and repeat 2 & 3.
4. Disconnect tubing and pour 30ml decontaminant solution into ends of tubing and all Leg Bag outlets.
5. Reconnect tubing and close Leg Bag outlet. Pour balance of solution into Leg Bag through the tubing and agitate for 30 seconds.
6. Drain solution into toilet, open outlet and hang Leg Bag in shaded place to air dry.
7. All parts must be completely dry before the next use.

***Decontamination Solution Preparation**

Option 1: Household Detergent/Soap. Dilute 5ml of detergent/soap in 150ml of warm tap water.

Option 2: Weak Vinegar Solution. Dilute 30ml of vinegar in 120ml of cold tap water.