

Protection, Maintenance and Cleaning / Disinfection of

HEINE HR[®] / HRP[®] / HR-C[®] Binocular Loupes

Protection

The Optics of the Loupes are delicate and should be protected against mechanical scratches caused by dust and / or dirt.

Please use the *Detachable Protective lenses* when using the Loupes.

Cleaning / Disinfection

Prior to Cleaning, remove the lens protectors.

The Loupes can be rinsed under running water. The Optics are water-resistant.

If residues have to be removed from the Optics and / or Housing a soft brush with plastic or natural bristles can be used.

Caution: Hard and /or metallic brushes can lead to scratches and in consequence to damage to the Optics.

Do not use any abrasive detergents for cleaning.

The Housing and Optics can be cleaned by wiping with a damp cloth. Use pure water or a pH-neutral or light soapy / alkaline solution.

For Cleaning, e.g. removing fingerprints from the Optics use a micro-fibre cloth.

Disinfection by spraying and / or wiping is permitted. For disinfection use only etergents which are released for Medical Devices made of Plastic.

Iso-Propylalcohol should only be used at a concentration below 70%.

Using of the HEINE Lens Cleaning Fluid

The HEINE Lens Cleaning Fluid can also be used for cleaning of the Optics and the Plastic Components of the Loupes. Spraying of the Loupes with this fluid is permitted, as the loupes are water-resistant.

Caution: The HEINE Lens Cleaning Fluid is not a disinfection solution.