

Melgisorb® Ag

Highly absorbent antimicrobial alginate dressing



Silver-containing alginate


- Provides sustained antimicrobial effect
- In vitro testing shows an antimicrobial effect from 4 hours up to 14 days

Alginate with CMC fibers


- Suitable for highly exuding wounds
- Designed for shallow and deep wounds
- Allows removal in one piece

- Excellent absorption for heavily exuding deep wounds
- Provides sustained antimicrobial effect
- High fibre strength allows removal in one piece


How to use Melgisorb® Ag

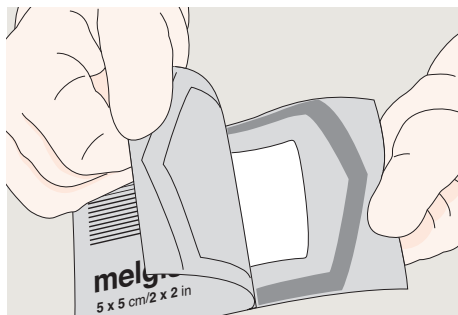

1. Debride and irrigate the wound site in accordance with standard protocols. Select a size of Melgisorb® Ag that is slightly larger than the wound. Cut or fold the dressing to fit the wound.

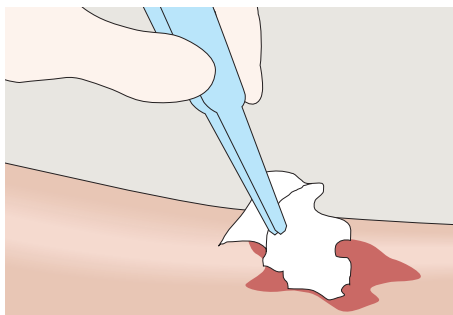

2. Loosely fill deep wounds, ensuring the dressing does not overlap the wound margins. Apply directly to the wound bed.

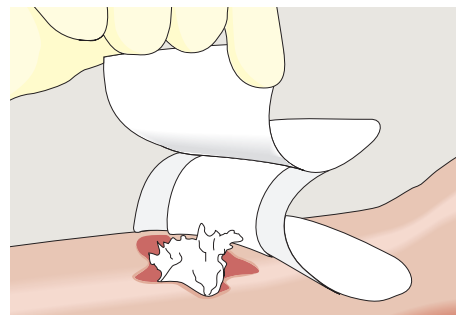

3. Cover and secure Melgisorb Ag with a secondary dressing.

Areas of use

Melgisorb Ag is indicated for the management of moderate to heavily exuding partial to full-thickness wounds such as:

- post-operative surgical wounds
- trauma wounds (dermal lesions, trauma injuries or incisions)
- leg ulcers
- pressure injuries
- diabetic ulcers
- graft and donor sites
- partial thickness burns

This product can also be used under compression bandages. As the product contains alginate, it may assist in supporting the control of minor bleeding in superficial wounds. Melgisorb Ag may be used in the management of infected wounds or wounds in which there is an increased risk of infection (under the medical supervision of a healthcare professional). Melgisorb Ag is indicated for external use only.

Note

Melgisorb Ag can remain in situ up to 7 days. Dressing change frequency will depend on the wound condition and the level of exudate. Initially, it may be necessary to change the dressing every 24 hours. If the wound appears dry, saturate the dressing with sterile saline solution prior to removal. Gently remove the dressing from the wound bed. Irrigate the wound site in accordance with standard protocols prior to application of a new dressing.

Contra-indications

Melgisorb Ag is not indicated for use on the following:

- individuals with a known sensitivity to alginates or silver
- to control heavy bleeding
- for direct application on dry or lightly exuding wounds.

For detailed information, please see Instructions for use



Assortment (sterile packed)

Art. No.	Size cm	Pcs/box
256055	5 x 5	10
256105	10 x 10	10
256155	15 x 15	10
256205	20 x 30	5
256605	3 x 44	10

www.molnlycke.com.au | www.molnlycke.co.nz

You will be required to input the following code **1FYrV** or your **AHPRA registration number** to access our websites.

Mölnlycke Health Care, 12 Narabang Way, Level 4, Belrose, NSW, 2085. Phone 1800 005 231. New Zealand Orders & Enquiries 0800 005 231. www.molnlycke.co.nz. The Mölnlycke, Melgisorb Ag name and logo are registered globally to one or more of the Mölnlycke Health Care Group of Companies. ©2022 Mölnlycke Health Care. All rights reserved. AUWC_000074_V1

