


# Melgisorb<sup>®</sup> Plus

Absorbent alginate dressing

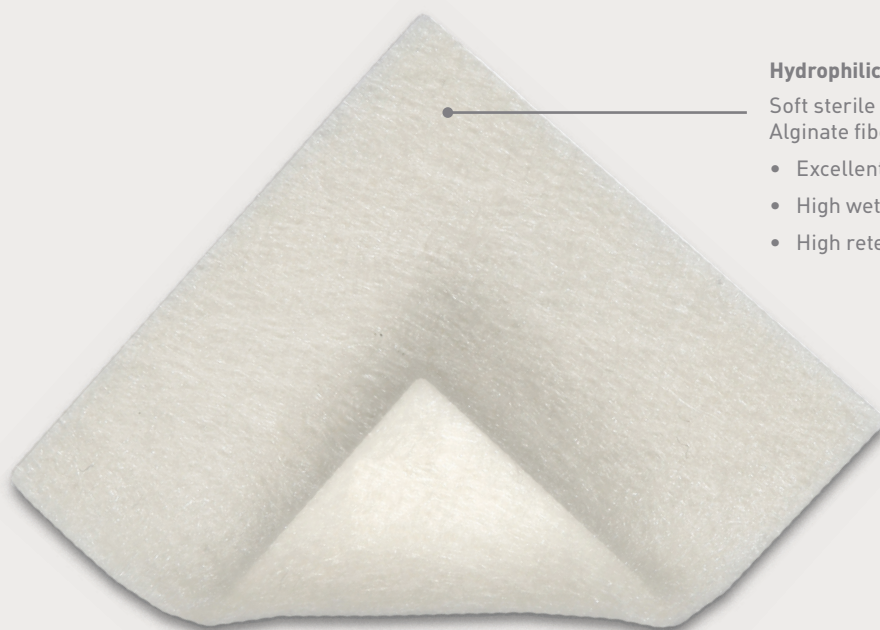


#### Hydrophilic alginate fibers

Soft sterile dressing made of Calcium Sodium Alginate fibers.


- Excellent absorption – for highly exuding wounds
- High wet strength – to allow removal in one piece
- High retention – to limit maceration

Melgisorb Plus is a highly absorbent dressing for a wide range of moderately to heavily exuding wounds.

## How to use Melgisorb® Plus

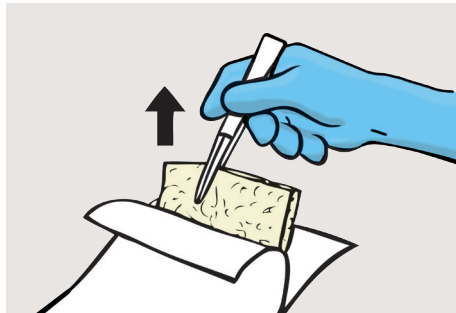


1. Select a size of Melgisorb Plus that is slightly larger than the wound. Pull the package open.

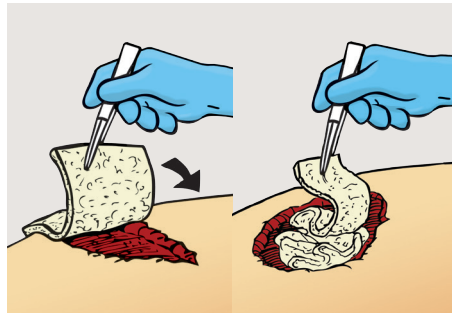

2. Apply a dry Melgisorb Plus product to the moist wound bed.

**Shallow wounds:** Choose the correct size of flat dressing to be able to cover the entire wound.

**Deep wounds and cavities:** Cut the appropriate length of the cavity dressing and pack the wound loosely.

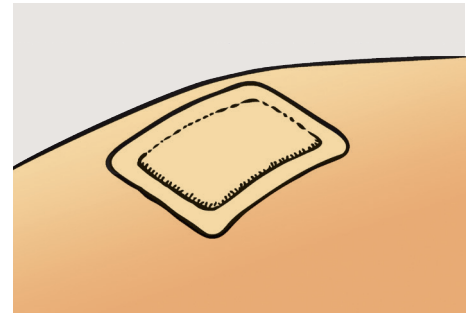

3. Cover and secure Melgisorb Plus with an appropriate secondary dressing such as Mepilex® Border Flex.

Compression therapy may be used in conjunction with Melgisorb Plus.

## How Melgisorb Plus works

Melgisorb Plus absorbs large amounts of wound exudate. When absorbing, the alginate fibres form a soft moist gel, creating a moist environment conducive to wound healing. The gel can easily be removed by irrigating with 0.9% saline.

## Benefits of Melgisorb Plus

- Highly absorbent
- Does not adhere to the wound site when gelled
- Suitable for cavity wounds
- Creates a gel on contact with exudate or solutions containing sodium ions (saline)
- Soft / pliable / conformable
- Can be cut to shape
- Non-sensitising

## Precautions

Melgisorb Plus is not indicated for dry wounds, third degree burns or surgical implantations.

If infection is suspected, follow local routines given by healthcare professionals.

Do not reuse. If reused, performance of the product may deteriorate and cross-contamination may occur.

## Intended Use

Melgisorb Plus is intended for a wide range of moderately to heavily exuding wounds, both infected and noninfected, such as:

- Pressure injuries
- Venous and arterial ulcers
- Diabetic ulcers
- Donor sites
- Post-operative wounds
- Dermal lesions and other external wounds inflicted by trauma

Configurations available both for flat and cavity wounds.



## Melgisorb Plus (Sterile packed)

Product Code	Size (cm)	Pcs/Box
252000	5x5	10
252200	10x10	10
252500	10x20	10
253500	3x45	5

[www.molnlycke.com.au](http://www.molnlycke.com.au) | [www.molnlycke.co.nz](http://www.molnlycke.co.nz)

You will be required to input the following code **1FtYrV** or your **APHRA registration number** to access our websites.

The Mölnlycke, Melgisorb and Mepilex names and logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies. ©2021 Mölnlycke Health Care. All rights reserved. Mölnlycke Health Care, 12 Narabang Way, Level 4, Belrose, NSW, 2085. Phone 1800 005 231. [www.molnlycke.com.au](http://www.molnlycke.com.au). New Zealand Orders & Enquiries 0800 005 231, [www.molnlycke.co.nz](http://www.molnlycke.co.nz). AUWC\_IM000417\_V2.

