



**AirFit™ P30i**  
Nasal pillows mask

**AirFit™ N30i**  
Nasal cradle mask



**This is for the active sleepers,**  
who go to bed when it suits them.  
For the night-time nomads  
who twist and turn  
then spring up to take on the day.



## Freedom to be themselves

The ResMed AirFit N30i and AirFit P30i give your patients the freedom to sleep in many positions and be close to their bed partner.


The discreet, minimalist design of both masks feature a convenient top-of-the-head tube design and self-adjusting SpringFit frame for a stable fit that lasts all night.<sup>1</sup>



## Streamline your business

The AirFit N30i and AirFit P30i both have a streamlined setup, designed to minimise fitting time and stock management. There are only two frame sizes that aim to deliver a good fit for the vast majority of patients, and the modular design of the AirFit N30i and AirFit P30i cushions means each size cushion for both masks can easily click into the same SpringFit frame. Simplify your logistics and help your patients feel comfortable and cared for.

## AirFit N30i and AirFit P30i components

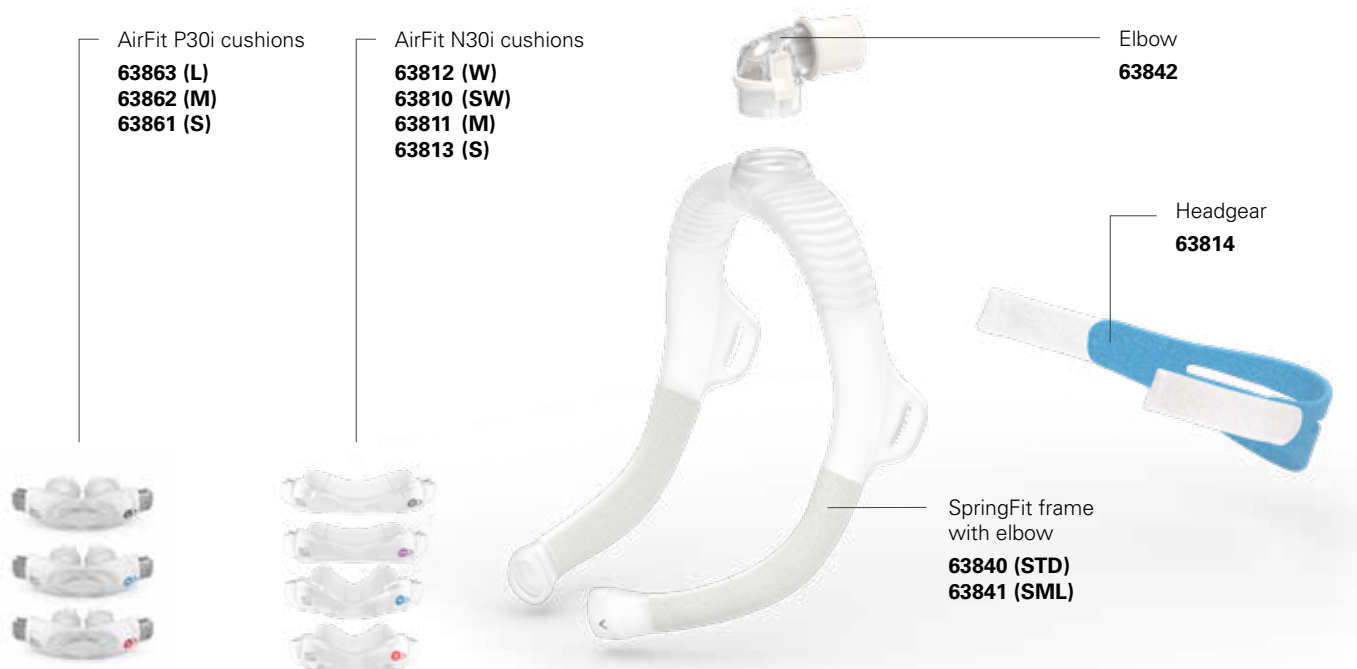

## Product codes

	ANZ
AirFit N30i Starter Pack: Small Includes: AirFit 30i small frame and headgear Cushions: S, M, SW	63826
AirFit N30i Starter Pack: Standard Includes: AirFit 30i standard frame and headgear Cushions: S, M, SW	63825
AirFit P30i Starter Pack: Small Includes: AirFit 30i small frame and headgear Cushions: S, M	63876
AirFit P30i Starter Pack: Standard Includes: AirFit 30i standard frame and headgear Cushions: S, M, L	63875

# Respond to your patients wants and needs

The tube-up design of the AirFitN30i and AirFit P30i was developed to be **adaptive for active sleepers.**

Both masks have been designed to let your patients sleep in many positions without compromising seal or comfort. The carefully engineered SpringFit frame, cushions and top-of-the-head design keep pace with every twist and turn. If they get up during the night, the quick-release elbow helps to keep life simple.

**SpringFit frame** adjusts to different facial profiles, helping provide an accurate fit the first time.

**Quick-release elbow** allows patients to easily disconnect mask from tubing with 360° rotation for greater freedom of movement.

**Soft sleeves** complement the streamlined mask profile to create a softer, more comfortable experience for the patient.


**AirFit N30i curved cushion design** cradles the shape of the nose and smoothly transitions from thicker side walls for added stability, to a thinner soft centre giving comfort where it's needed most.


**AirFit P30i pillows cushion** are designed to be interchangeable with the AirFit N30i nasal cradle cushion, offering different options.

**Septum bridge membrane** ensures an intuitive fit and provides a stable seal<sup>2</sup> all the way to the tip of the nose through the night.



**QuietAir™** vent gently diffuses exhaled air, designed to minimise noise and disturbance.

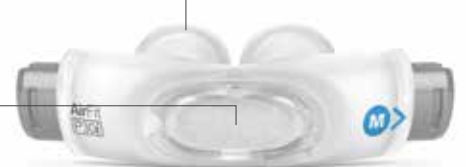

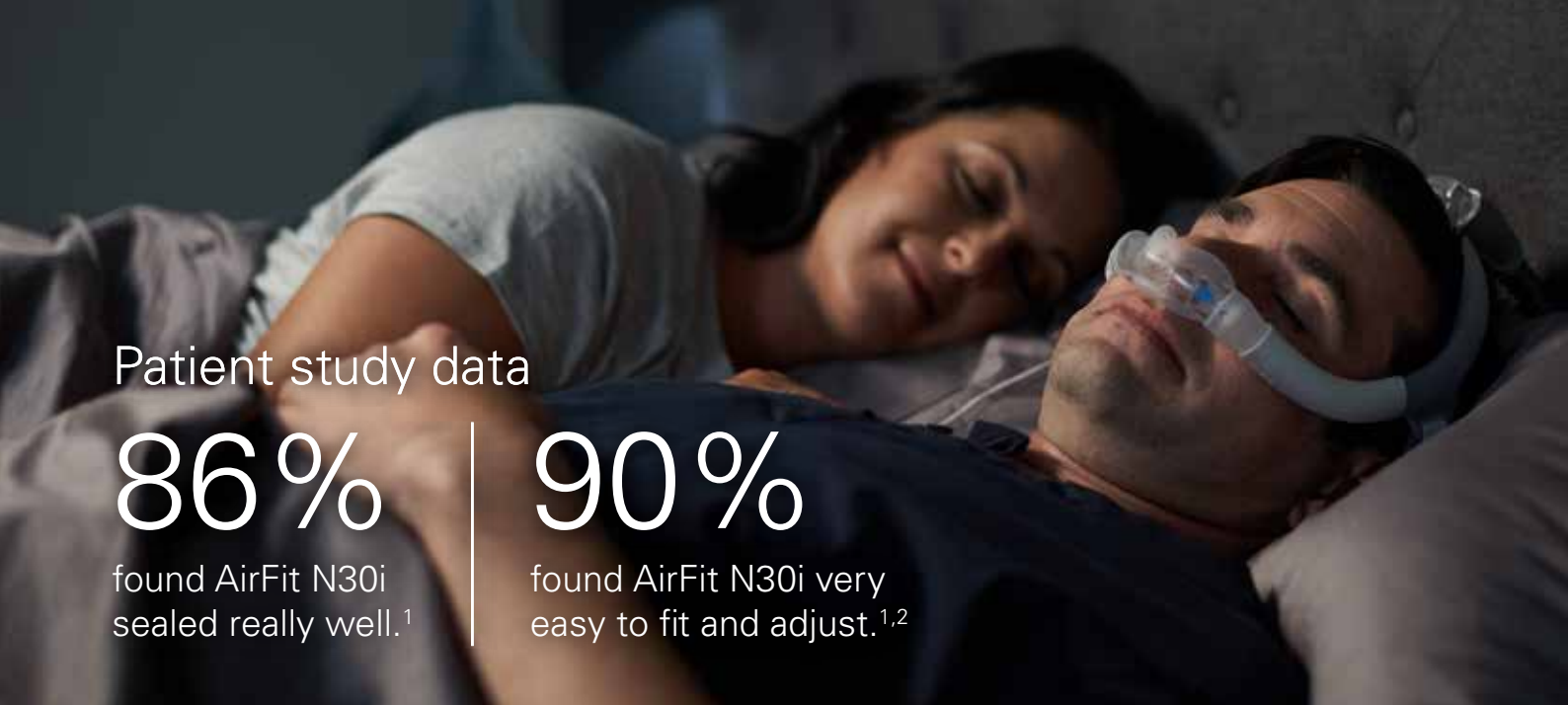

## Patient study data

# 86%

found AirFit N30i sealed really well.<sup>1</sup>


# 90%

found AirFit N30i very easy to fit and adjust.<sup>1,2</sup>

## Sizing tips

**AirFit N30i has four interchangeable cushion sizes** - small, medium, small wide and wide.

We recommend that you refer to the mask fitting template, included in each Starter Pack, to gauge which size is the most suitable for your individual patients.

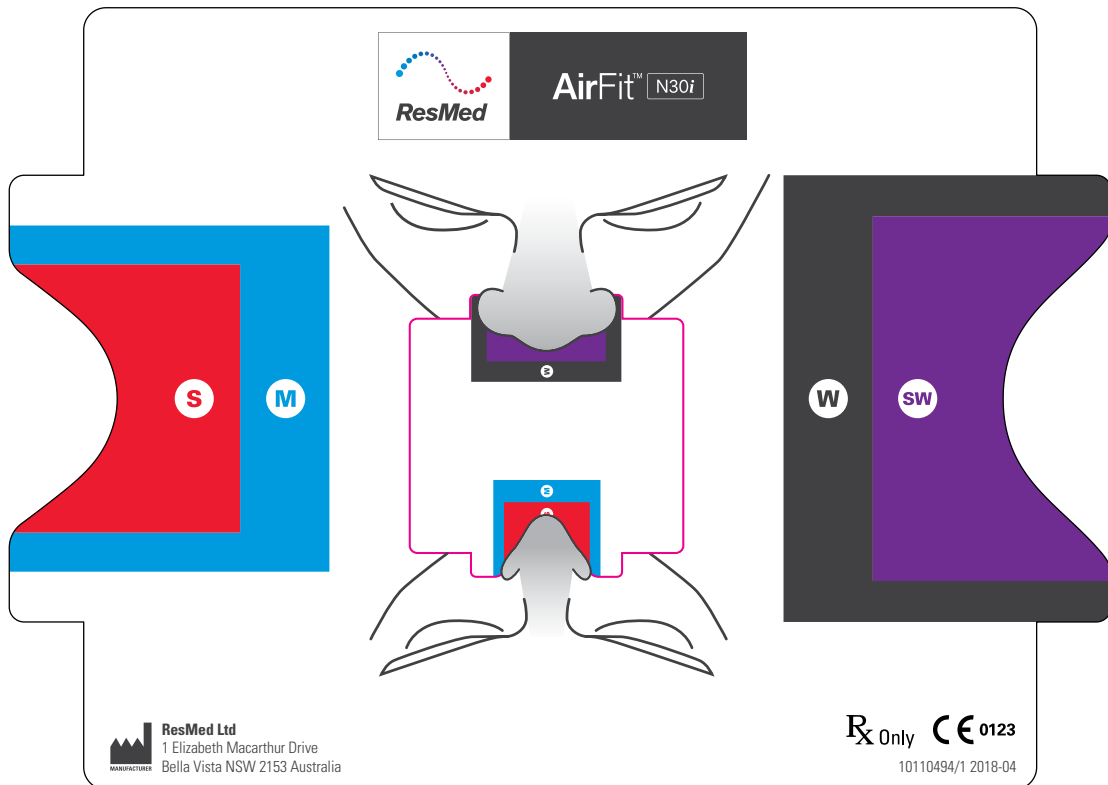

**AirFit P30i has three interchangeable cushion sizes** - small, medium and large.

We recommend patients try each size pillows cushion to determine which is most comfortable.

1 ResMed external 7 days clinical study of existing 24 ResMed and non-ResMed patients, conducted between 21/02/2018 - 12/03/2018. Data on file; ID A4320645.  
2 ResMed external fitting study of existing 37 ResMed and non-ResMed patients, conducted between 09/04/2018 - 20/04/2018. Data on file; ID A4327753.





Nasal cradle mask




Nasal pillows mask

## Fitting guide for AirFit N30i and AirFit P30i




- Pull the frame and headgear over your head until the elbow is positioned on top and the headgear is behind your head.

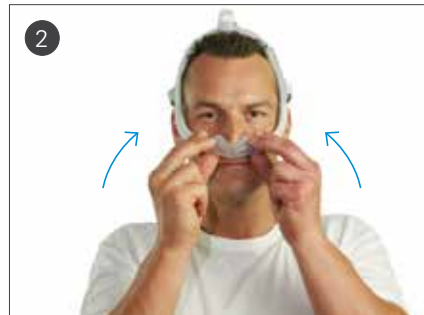

- Place the cushion under your nose
- For the AirFit P30i mask, align the pillows into your nostrils.



- Tighten or loosen the headgear until the cushion fits comfortably under your nose.

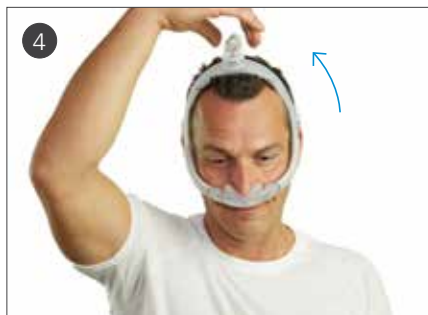

- Squeeze the side buttons on the elbow and detach from the frame.



- Connect the air tubing from your device to the elbow.



- Reattach the elbow to the frame.
- The mask is now ready to use.

Refer to the user guide for further instructions.



### Additional options

- For patients with longer hair, we recommend feeding their hair through the back of the headgear. If tied in a pony tail, it should sit high enough to fit comfortably through the back of the headgear.
- Refer to the user guide for further instructions and complete labeling information including indications, contraindications, warnings and precautions.

**AirFit™** N30i

**AirFit™** P30i

This is for them.



**ResMed Ltd**  
1 Elizabeth Macarthur Drive  
Bella Vista NSW 2153 Australia  
+61 (2) 8884 1000 or 1800 658 189

AirFit is a trademark and/or registered trademark of the ResMed family of companies. For patent information, see ResMed.com/ip.  
Specifications may change without notice. © 2019 ResMed Ltd. 10111226/1 2019-02

[ResMed.com/AirFit30Series](https://ResMed.com/AirFit30Series)



**ResMed**