

# A CLINICALLY PROVEN COMBINATION\*

\*Collagen and Oxidised Regenerated Cellulose

## WHAT IS IT?

PROMOGRAN® is comprised of 45% oxidized regenerated cellulose (ORC) and 55% collagen in a sterile, freeze dried composite.

PROMOGRAN PRISMA™ is comprised of 44% oxidized regenerated cellulose (ORC), 55% collagen and 1% silver-ORC in a sterile, freeze dried composite.


### OPTIMUM WOUND HEALING ENVIRONMENT

PROMOGRAN® and PROMOGRAN PRISMA™ maintain an optimal wound healing environment<sup>1-6</sup>. This environment is conducive to granulation tissue formation, epithelisation and rapid wound healing.



### CLINICALLY PROVEN

The efficacy of PROMOGRAN® and PROMOGRAN PRISMA™ is supported by a large body of clinical evidence, including 10 published RCTs<sup>7-16</sup>.

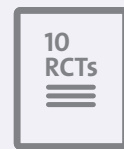

### COST-EFFECTIVENESS

In clinical practice the sequential treatment with PROMOGRAN PRISMA™ and PROMOGRAN® has been found to be a cost-effective treatment for chronic wounds<sup>17</sup>.



## PROMOGRAN® & PROMOGRAN PRISMA™



# WHAT WOUNDS CAN I USE IT ON? ANY WOUNDS HEALING BY SECONDARY INTENTION INCLUDING...



Leg ulcers



Pressure ulcers



Diabetic ulcers



Surgical wounds

## HOW DO I PREPARE THE WOUND?

Before treatment, devitalised tissue, such as dry necrotic tissue, must first be removed by sharp, enzymatic or autolytic debridement.

These wounds still require debridement prior to PROMOGRAN® or PROMOGRAN PRISMA™ treatment



These wounds are now ready for closing with PROMOGRAN® or PROMOGRAN PRISMA™



## IS THERE ANYTHING ELSE I SHOULD KNOW?


- 1** PROMOGRAN® and PROMOGRAN PRISMA™ can be moulded to fit the wound shape, even a sinus
- 2** PROMOGRAN® and PROMOGRAN PRISMA™ can be used under compression therapy
- 3** PROMOGRAN® and PROMOGRAN PRISMA™ have known haemostatic properties
- 4** After initial treatment, retreat the wound with PROMOGRAN® or PROMOGRAN PRISMA™ up to every 72 hours depending on the amount of exudate

# USER GUIDE

## BEFORE APPLICATION


Refer to the instructions for use. Before treatment, necrotic tissue must first be removed by debridement. Prepare the wound according to appropriate wound management protocol.

### Dressing preparation

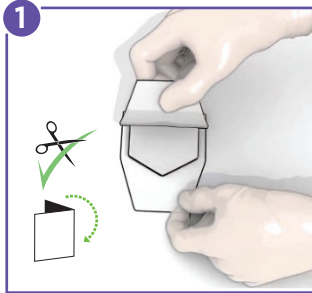


1. Peel back the tray cover and leave dressing in tray.
2. If necessary, **PROMOGRAN®** / **PROMOGRAN PRISMA™** can be cut using clean scissors or folded to fit the wound bed.
3. For a small or deep exuding wound, cut the dressing prior to application or pack dressing into the wound.

### Dressing application

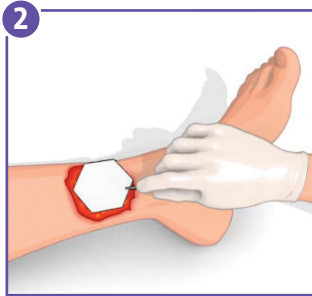


1. If the wound bed is wet apply the dressing directly.
2. If the wound is dry moisten the dressing with saline.

### Secondary dressing application

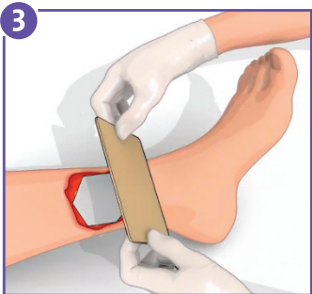

1. Cover with a secondary dressing such as a non adherent layer like **ADAPTIC TOUCH™** or a foam dressing such as **TIELLE®**. Choose the secondary dressing based on the level of exudate.
2. Apply any secondary cover or retention/compression bandage.

### Dressing change and removal

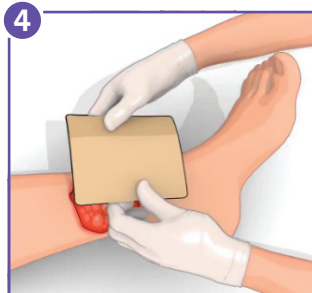

1. Carefully remove the secondary dressing. It is not necessary to remove any residual **PROMOGRAN®** / **PROMOGRAN PRISMA™** dressing during dressing changes.

## HOW IS PROMOGRAN PRISMA™ DIFFERENT TO PROMOGRAN®?

- **PROMOGRAN PRISMA™** contains a compound of silver and ORC
- **PROMOGRAN PRISMA™** has nominally twice the amount of Collagen/ORC material as the same size of **PROMOGRAN®**
- **PROMOGRAN PRISMA™** contains ionically bound silver, a well-known antimicrobial agent which provides antimicrobial protection against bacteria and infection<sup>6, 8, 18</sup>

Good to know...

It is **not** necessary to remove any residual **PROMOGRAN®** / **PROMOGRAN PRISMA™** left in the wound at dressing change.

### PROMOGRAN PRISMA™

Size	Quantity per Carton	Product code
28cm <sup>2</sup>	10	PS2028
123cm <sup>2</sup>	10	PS2123

### PROMOGRAN®

Size	Quantity per Carton	Product code
28cm <sup>2</sup>	10	M772028
123cm <sup>2</sup>	10	M772123



An Acelity Company

For more information, contact your KCI representative or Customer Service

1300 524 822

1800 524 822

[www.systagenix.com.au](http://www.systagenix.com.au)



References:

1. Hart, J. et al. The role of oxidized regenerated cellulose / collagen in wound repair: effects in vitro on fibroblast biology and in vivo in a model of compromised healing. *Int J Biochem Cell Biol* 2002;34:1557-1570.
2. Cullen, B. et al. Modulation of the chronic wound environment; an in vitro evaluation of advanced wound therapies Poster, SAWC 2007
3. Cullen, B. et al. Collagen ORC rebalances the wound environment. Poster WHS, 2003
4. Cullen, B. et al. Use of oxidized regenerated cellulose in facilitating wound healing. Poster, EWMA 2010.
5. Cullen B. et al. ORC/Collagen/Silver-ORC matrix promotes cell growth in the presence of chronic wound fluid. Poster, SAWC 2007
6. Cullen B. et al. ORC/Collagen Matrix containing silver controls bacterial bioburden while retaining dermal cell viability. Poster, SAWC 2006.
7. Wollina, U. et al. Some Effects of a Topical Collagen-Based Matrix on the Microcirculation and Wound Healing in Patients With Chronic Venous Leg Ulcers: Preliminary Observations. *Low Extr WOUNDS* 2005, vol 4(4):214-224.
8. Gottrup, F. et al. Randomized controlled trial on collagen/oxidized regenerated cellulose/silver treatment. *Wound Rep Reg* 2013, 21:1-10
9. Nisi G. et al. Use of a protease-modulating matrix in the treatment of pressure sores. *Chir Ital* 2005, vol. 57(4):465-468
10. Lazaro-Martinez, J. L. et al. Randomized comparative trial of a collagen/oxidized regenerated cellulose dressing in the treatment of neuropathic diabetic foot ulcers. *F.R. Circ. Esp.* 2007, 82(1), 27-31
11. Veves, A. et al. A Randomized, Controlled Trial of Promogran (a Collagen Oxidized Regenerated Cellulose Dressing) vs Standard Treatment in the Management of Diabetic Foot Ulcers. *Arch. Surg* 2002, vol. 137:822-827
12. Vin, F. et al. The healing properties of Promogran in venous leg ulcers. *J. Wound Care* 2002; 11(9):335-41.
13. Lobmann, R. et al. Expression of matrix metalloproteinases and growth factors in diabetic foot wounds treated with a protease absorbent dressing. *J Diabetes Complications* 2006 ; 20(5) : 329-335
14. Ulrich, D. et al. Effect of oxidized regenerated cellulose/collagen matrix on proteases in wound exudate of patients with diabetic foot ulcers. *J Wound Ostomy Continence Nurs.* 2011 38(5) 1-7
15. Smeets, R. et al. Effect of oxidised regenerated cellulose/collagen matrix on proteases in wound exudate of patients with chronic venous ulceration. *Int. Wound J.* 2008, 5:195-203
16. Kakagia, D.D. et al. Synergistic action of protease-modulating matrix and autologous growth factors in healing of diabetic foot ulcers. A prospective randomized trial. *J Diabetes Complications* 2007; 21(6): 387-91.
17. Snyder, R. et al. A Retrospective Study of Sequential Therapy with Advanced Wound Care Products versus Saline Gauze Dressings: Comparing Healing and Cost. *Ostomy Wound Management* 2010;56(11A):9-15
18. Simmons, R. et al. Effect of collagen ORC silver on bacterial proteases. Poster, CAWC 2013

© Systagenix Wound Management 2014. Brands marked with ® or ™ are trademarks of Systagenix. All other products referenced herein are acknowledged to be trademarks of their respective owners.