SIGVARIS AG
Gröblistrasse 8
9014 St. Gallen / Switzerland
Tel: +41 71 272 40 00
www.sigvaris.com

SIGVARIS is a registered trademark of SIGVARIS AG,
St. Gallen/Switzerland, in many countries worldwide.
© 2016 Copyright by SIGVARIS AG, St. Gallen/Switzerland

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Your personal assistant for donning and doffing compression stockings for arms and legs.

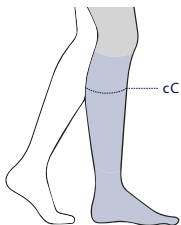
You can use SIMON to don and doff your compression stocking quickly, effortlessly, and with very little exertion. It can even be used if you have limited mobility. SIMON is always there when you need it. It will accompany you on trips and can be easily stowed when it's not needed.

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
X-small	25-29 cm
Small	30-38 cm
Medium	38-46 cm
Large	46-54 cm

cC: Wadenumfang (kräftigste Stelle) / Widest circumference of the calf /
Circonférence au plus fort du mollet / Circonferenza polpaccio (punto più forte)

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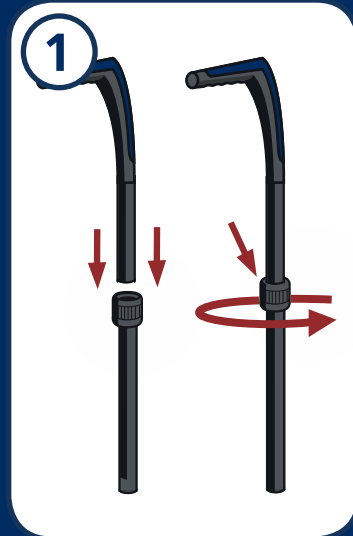
TIP: Do not unscrew the tube pieces again after assembly.

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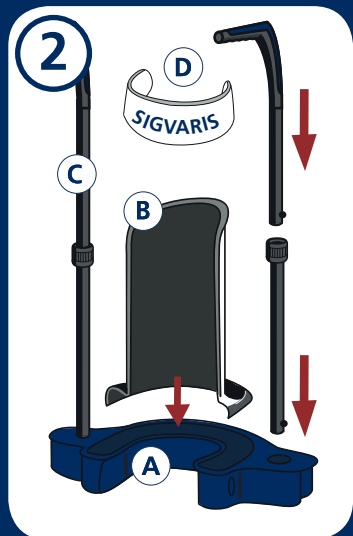
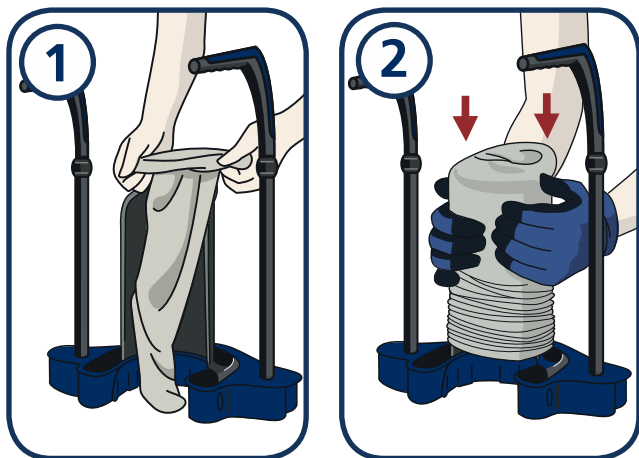
1. Pull the stocking over the donning shovel up to the centre of the heel.
The heel should point toward your body.

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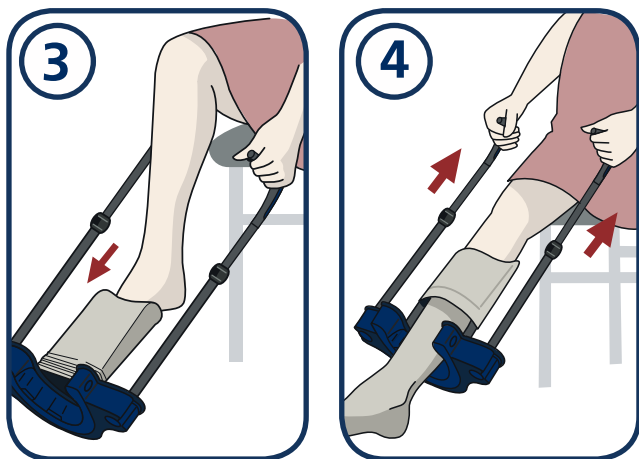
Press the buttons to remove the black donning shovel.

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
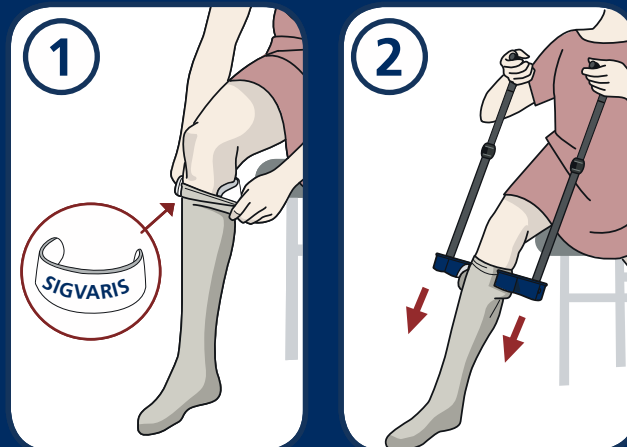
1. Place the ring below the hollow of your knee underneath the stocking.

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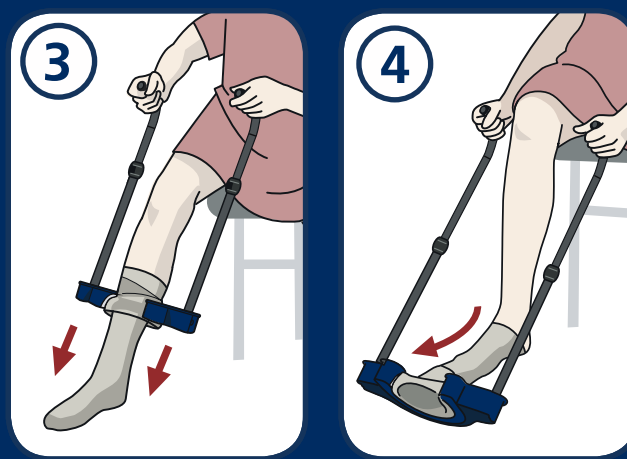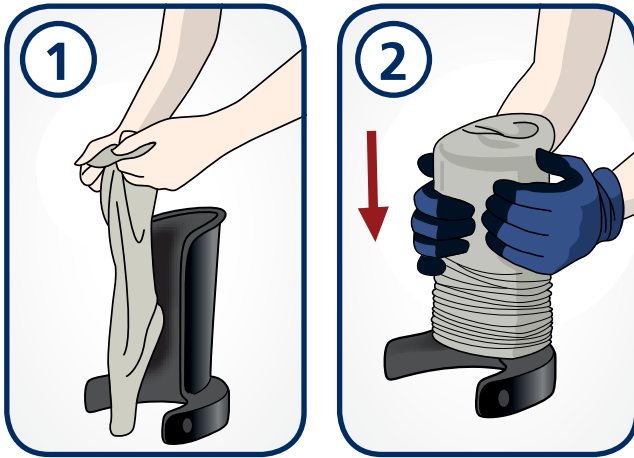
1+2: Pull the stocking over the donning shovel up to the centre of the heel. The heel should point toward your body.

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
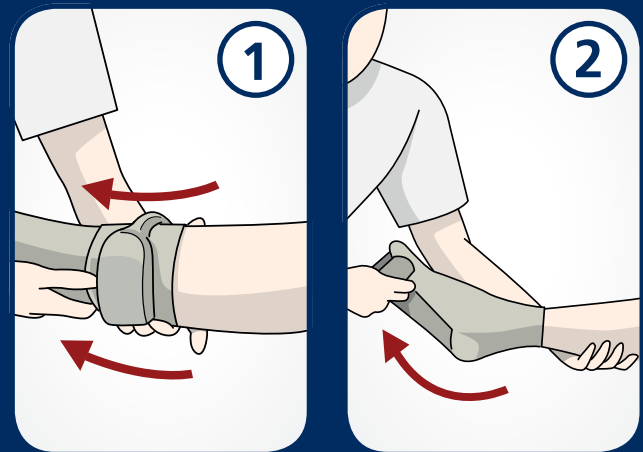
1+2: Fold the stocking over the ring below the hollow of your knee and pull the stocking and ring downwards.

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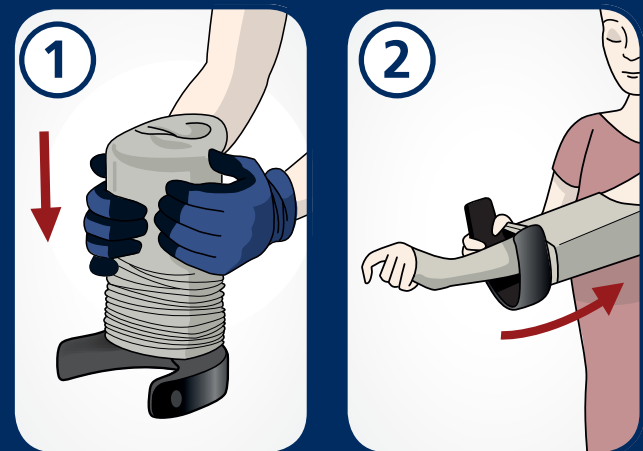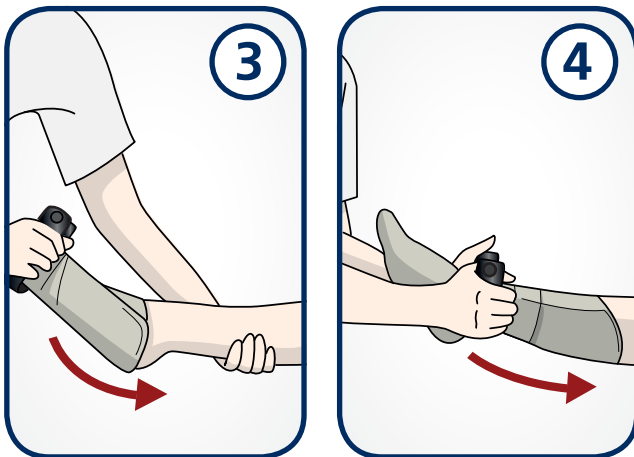1+2: Pull the stocking over the donning shovel up to the base of your hand.

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