



Product Instructions and Care

Ulcer x

Instructions

- If possible use special SIGVARIS Rubber Gloves to don socks or stockings
- Avoid sharp items and be careful of fingernails and toenails
- Remove rings before applying
- Do not apply oily skin care lotion on your legs before wearing the product

Care

- Wash daily in a linen bag or wash by hand in cold/lukewarm water using the SIGVARIS Washing Solution or any mild detergent (suitable for washing delicates)
- Do not dry on a direct heat source
- Do not use fabric softener
- Do not tumble dry
- Do not dry clean
- Do not iron

How to apply the Ulcer X

For instructions on how to put on your compression stocking please click on the video link below:

[How to Put on your compression stocking with the Sigvaris Doff N Donner](#)