

CLEAN & KILL STOP THE VICIOUS CYCLE OF INFECTION

URGO
MEDICAL
Healing people®



UrgoClean Ag

The only anti-biofilm silver dressing
that provides complete and continuous
cleaning action in the wound


ADVANCES IN
INFECTION AND
BIOFILM



WOUND INFECTION CREATES A VICIOUS CYCLE THAT CAUSES TISSUE DAMAGE & DELAYED HEALING

Decreases wound healing by sustaining Biofilm & microbial dissemination

Stimulates inflammation (slough, exudate production), prevents wound healing & limits action of silver dressings

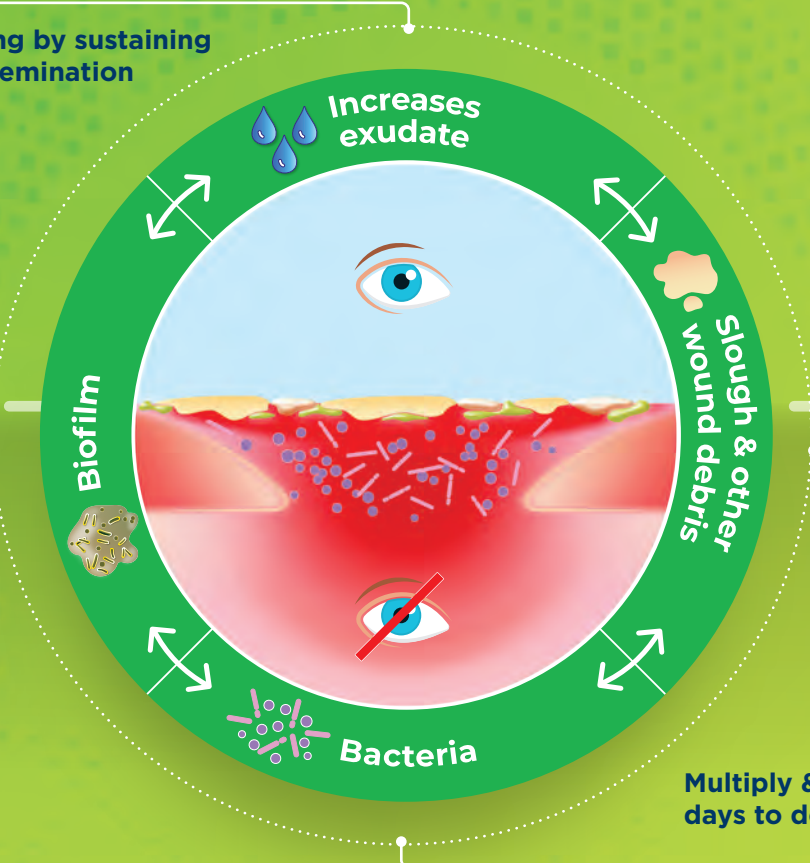

Favours microbial proliferation & biofilm formation

Multiply & form biofilm in a few days to delay wound healing


A COMBINED ANTIMICROBIAL & COMPLETE CLEANING ACTION IS NECESSARY TO EFFECTIVELY FIGHT LOCAL INFECTION AND RESTORES HEALING PROCESS

Stop the vicious cycle of wound infection



Use UrgoClean Ag
For all wounds at risk or with signs of local infection, from the debridement stage

THE ONLY ANTI-BIOFILM SILVER DRESSING THAT PROVIDES COMPLETE AND CONTINUOUS CLEANING ACTION IN THE WOUND

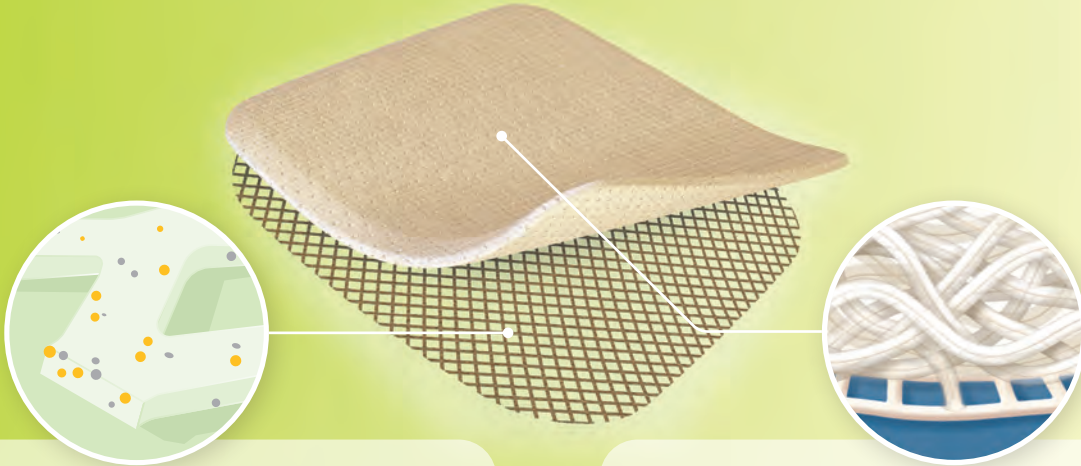

TLC-Ag Technology

- ▶ **Fast and effective anti-biofilm action**
 - ✓ 99.99% biofilm reduction achieved in just 24h¹
 - ✓ Blocks biofilm reattachment for up to 7 days²
 - ✓ Super anti-biofilm efficacy compared to AquacelAg+ Extra and Cutimed Sorbact^{1,3}
- ▶ **Proven antimicrobial efficacy**
 - ✓ Fast action: from 30 minutes on main strains²
 - ✓ Works against bacterial strains resistant to antibiotics (e.g. MRSA, VRE)²
- ▶ **TLC (Technology Lipido-Colloid)⁴**
 - ✓ No pain at dressing changes
 - ✓ Will not damage newly formed tissue
 - ✓ Non-adherence to the wound bed
 - ✓ Restores the healing process

Polyabsorbent Fibres

- ▶ **Complete and continuous cleaning action**
 - ✓ High affinity to vertically wick, trap and retain wound debris such as sloughy residues, microbes and biofilm⁹
- ▶ **Strong acrylic core**
 - ✓ Easy one-piece removal
- ▶ **Maintains a clean wound**
 - ✓ Optimising the silver efficacy in the wound⁹



Can be cut



Can be used under compression




Moderate absorption



Can be used with other dressings

Antimicrobial activity

Silver ions provide fast & broad spectrum antimicrobial efficacy⁵

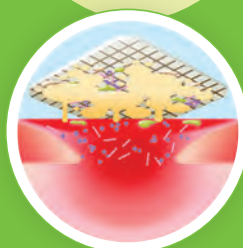

Anti-biofilm activity

Combined cleaning and antimicrobial action defeats biofilms and prevent their reattachment³




Complete & continuous cleaning action


Polyabsorbent fibres remove slough and exudate

UrgoClean Ag is indicated for all wounds at risk or with signs of local infection from the debridement stage

LEG ULCER

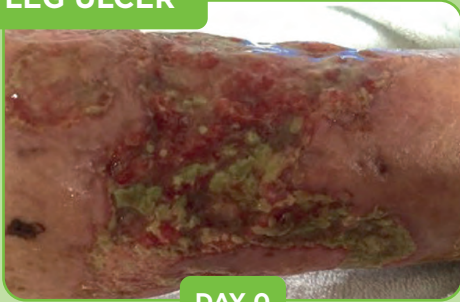

DAY 0

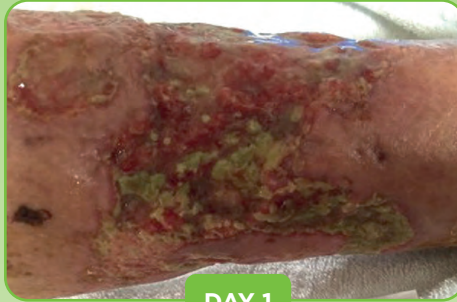

DAY 1



DAY 14

DIABETIC FOOT ULCER



DAY 1



DAY 14

TRAUMATIC WOUND



DAY 1



DAY 3




DAY 14

BURN WOUND



DAY 2

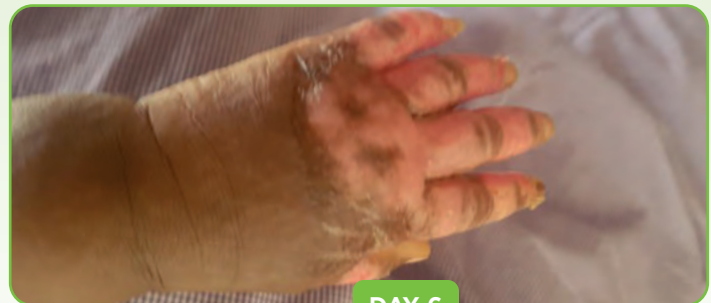

DAY 6

Proven in clinical trials...

UTAG⁷ RCT versus equivalent neutral dressings (superiority vs hypothesis)
Venous leg ulcers presenting inflammatory signs suggesting a heavy bacterial colonization
102 patients

EARTH⁸ RCT vs Aquacel[®] Venous or mixed aetiology leg ulcers
159 patients

URGOCLEAN AG⁴ clinical study
Venous or mixed aetiology leg ulcers and pressure ulcer presenting inflammatory signs suggesting a heavy bacterial colonization
37 patients

Cohort studies

URGOCLEAN AG⁹ observational study
1,050 Chronic, 876 Acute and 339 Unclassified.
Wounds at risk or with signs of local infection.
2,270 patients

Cases series or studies

...confirmed in real life

In Vitro studies

ANTIBACTERIAL ACTIVITY TEST⁵
MRSA and *P. aeruginosa*
Contact time: 24h

ANTI-BIOFILM ACTIVITY TEST⁶
Mature 24 hours old biofilm *S. aureus* of
Contact time: 1 day, 2 days, 4 days and 7 days without dressing change


BROAD-SPECTRUM ANTIMICROBIAL TEST¹⁰
36 bacterials strains and 4 yeast strains Contact time: 30 min and 24h

COMPARATIVE ANTI-BIOFILM ACTIVITY TEST¹
vs Aquacel[®] Ag+Extra
Mature 24 hours old biofilm of *S. aureus* and *P. aeruginosa* Contact time: 24hr



COMPARATIVE ANTI-BIOFILM ACTIVITY TEST¹⁰
vs Cutimed[®] Sorbact[®]
Mature 24 hours old biofilm of *S. aureus* and *P. aeruginosa* Contact time: Day 2



Superior anti-biofilm efficacy compared to other antimicrobial dressings thanks to UrgoClean Ag combined action

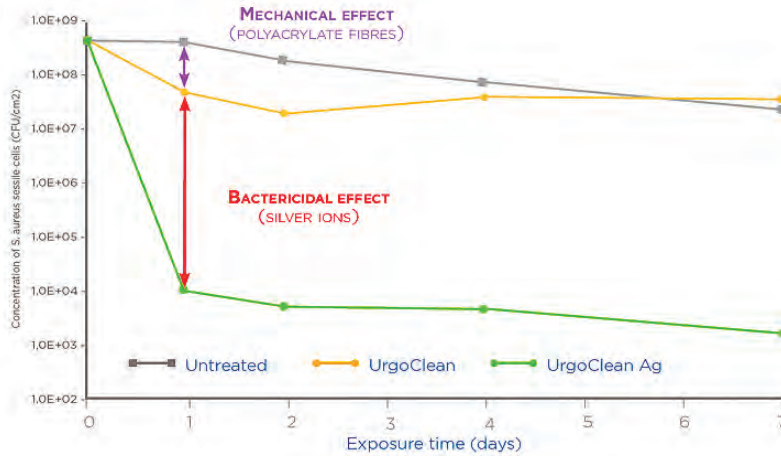


- UrgoClean Ag possesses an anti-biofilm activity against mature biofilms⁶.
- Anti-biofilm activity was maintained for 7 days with reduction values superior to 3 logs (reduction of biofilm superior to 99.90%).

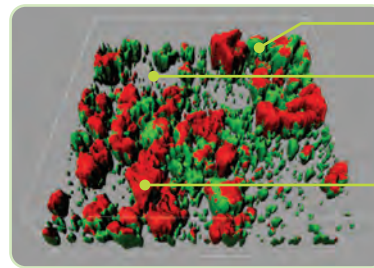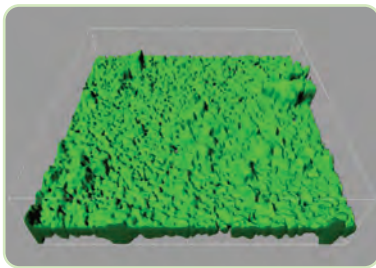


- Biofilm
- Biofilm destructuring (synergistic effect of Ag⁺ and fibres)
- Dead cells (bactericidal activity of Ag⁺)


UrgoClean Ag is 50 times stronger on MRSA Mature Biofilm¹¹

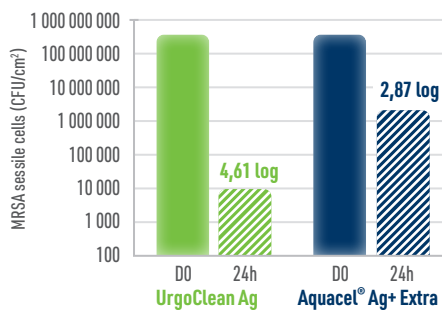


UrgoClean Ag is 100 times stronger on P. aeruginosa Mature Biofilm¹

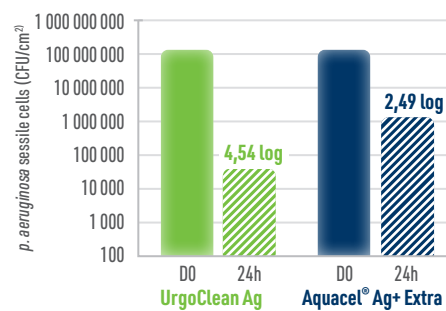

Biofilm of S. aureus¹⁰

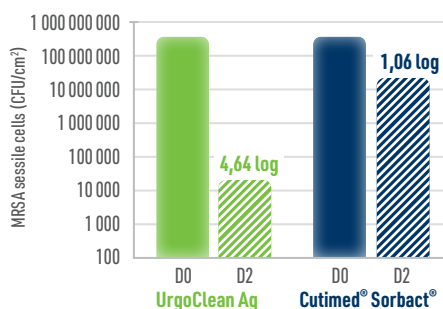

Biofilm of P. aeruginosa¹⁰

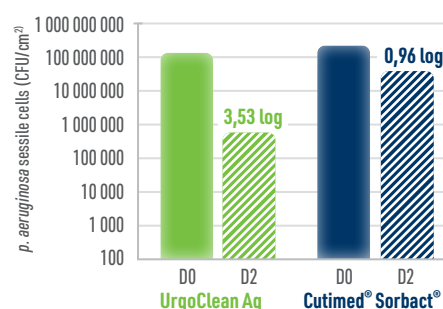


Earth (RCT) Study


Evaluation of the efficacy, safety & acceptability of the **UrgoClean** dressing vs the Aquacel® dressings in the local management of venous or mixed, predominantly venous leg ulcers

Methodology







- Randomised Controlled Trial (RCT)
- Multicentre (France, Germany, UK)
- 159 patients
- 6 weeks of treatment
- Venous leg ulcers with $\geq 70\%$ of wound surface area covered with slough
- Moderately to highly exuding ulcer requiring the use of an absorbent dressing
- Effective venous compression along with the dressing every day during the study

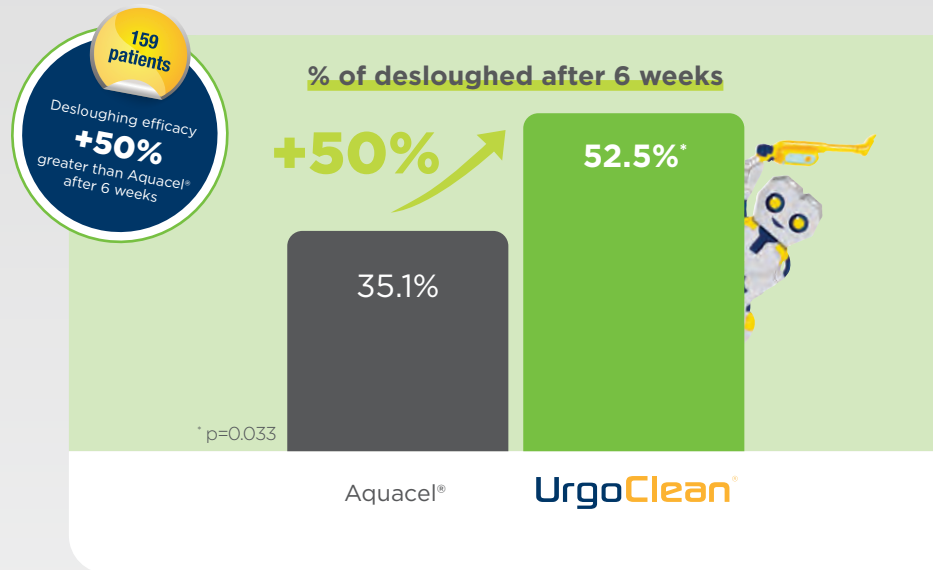

Trapping of slough

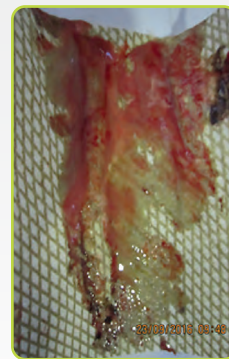

Trapping of slough

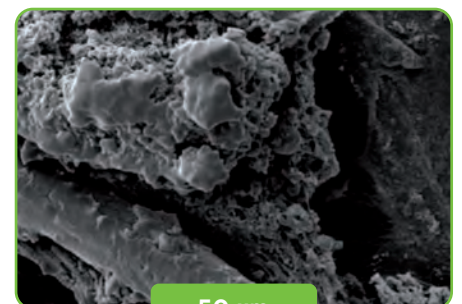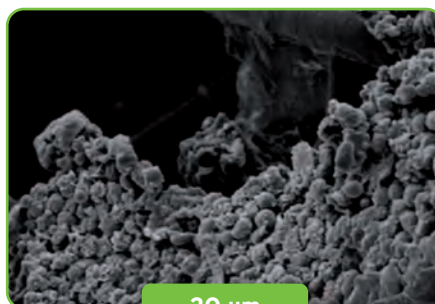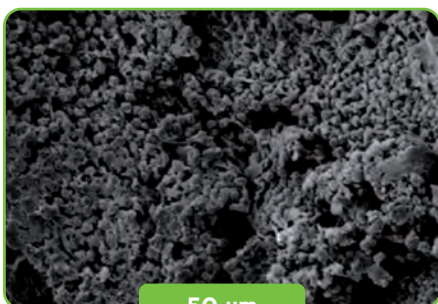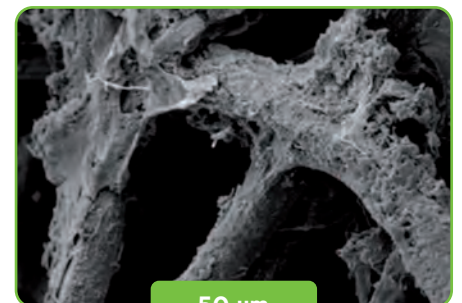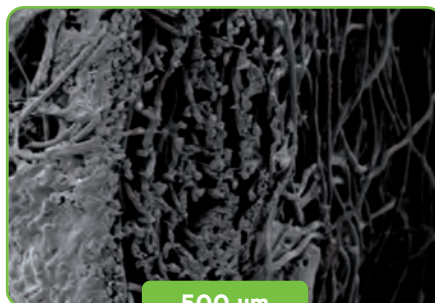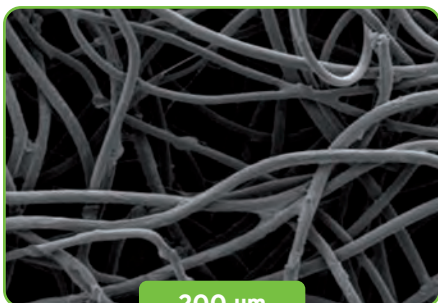

RISK FACTORS FOR WOUND INFECTION MAY BE RELATED

CHARACTERISTICS OF THE INDIVIDUAL

- Diabetes, prior surgery
- Chemotherapy/ radiation therapy
- Conditions associated with hypoxia or poor tissue perfusion
- Immune system disorders
- Malnutrition
- Nasal carriage of S aureus

WOUND FEATURES

- Contamination
- Delayed treatment
- Signs of local infection
- Operative factors
- Wound duration, wound size (width, depth)
- Anatomic location
- Impaired tissue perfusion
- Necrotic tissue

ENVIRONMENT

- Hospital
- Hygiene
- Inadequate management of moisture/exudate/oedema/ pressure offloading



UrgoClean Ag



Moderate absorption



Can be cut



Can be used with other dressing



Can be used under compression



Code	Size	No. per box
100490	6x6cm	10
100491	10x10cm	10
100492	15x20cm	5

- Percival S. L. (2018). Restoring balance: biofilms and wound dressings. *Journal of wound care*, 27(2), 102-113. <https://doi.org/10.12968/jowc.2018.27.2.102>
- Percival, S. L., & Suleman, L. (2015). Slough and biofilm: removal of barriers to wound healing by de-sloughing. *Journal of wound care*, 24(11), 498-510. <https://doi.org/10.12968/jowc.2015.24.11.498>
- Desroche N. et al. (2017). Comparison of in vitro anti-biofilm activities of a new poly-absorbent dressing with a silver matrix and a silver containing CMC dressing. Poster EWMA, May, 2017
- Dalac, S., Sigal, L., Addala, A., Chahim, M., Faivre-Carrere, C., Lemdjadi, Z., & Bohbot, S. (2016). Clinical evaluation of a dressing with poly absorbent fibres and a silver matrix for managing chronic wounds at risk of infection: a non comparative trial. *Journal of wound care*, 25(9), 531-538. <https://doi.org/10.12968/jowc.2016.25.9.531>
- Desroche N. et al. (2016). Characterization of the antimicrobial spectrum and anti-biofilm activity of a new silver-containing dressing with poly-absorbent fibres and antimicrobial silver matrix. Poster EWMA, May 2016
- Desroche, N., Dropet, C., Janod, P., & Guzzo, J. (2016). Antibacterial properties and reduction of MRSA biofilm with a dressing combining polyabsorbent fibres and a silver matrix. *Journal of wound care*, 25(10), 577-584. <https://doi.org/10.12968/jowc.2016.25.10.577>

- Lazareth, I., Meaume, S., Sigal-Grinberg, M. L., Combemale, P., Guyadec, T. L., Zagnoli, A., Perrot, J. L., Sauvadet, A., & Bohbot, S. (2008). The Role of a Silver Releasing Lipido-colloid Contact Layer in Venous Leg Ulcers Presenting Inflammatory Signs Suggesting Heavy Bacterial Colonization: Results of a Randomized Controlled Study. *Wounds : a compendium of clinical research and practice*, 20(6), 158-166.
- Meaume, S., Dissemond, J., Addala, A., Vanscheidt, W., Stücker, M., Goerge, T., Perceau, G., Chahim, M., Wicks, G., Perez, J., Tacca, O., & Bohbot, S. (2014). Evaluation of two fibrous wound dressings for the management of leg ulcers: results of a European randomised controlled trial (EARTH RCT). *Journal of wound care*, 23(3), 105-116. <https://doi.org/10.12968/jowc.2014.23.3.105>
- Dissemond, J., Dietlein, M., Neßeler, I., Funke, L., Scheuermann, O., Becker, E., Thomassin, L., Möller, U., Bohbot, S., & Münter, K. C. (2020). Use of a TLC-Ag dressing on 2270 patients with wounds at risk or with signs of local infection: an observational study. *Journal of wound care*, 29(3), 162-173. <https://doi.org/10.12968/jowc.2020.29.3.162>
- Desroche N., et al. (2017) Evaluation of in vitro anti-biofilm activities of two dressings with poly-absorbent dressing fibres and a DACC coated dressing. Poster EWMA, May 2017
- Desroche, N., et al. (2017). Evaluation of the anti-biofilm activity of a new poly-absorbent dressing with a silver matrix* using a complex in vitro biofilm model. Poster Wounds UK 2017.