

**UrgoStart® Plus closes
wounds sooner***
from day one until
complete healing^{1,2}



NICE
Recommended

PRACTICAL GUIDE



UrgoStart® plus
Treatment Range

URGO
MEDICAL
Healing people®


A simple and efficient treatment for **all leg ulcer, diabetic foot ulcer** and **pressure ulcer** patients

Effective

Reduces healing time^{1,2} right from the start, with the interactive matrix

Simple

Use from day 1 to complete healing, at all healing phases*

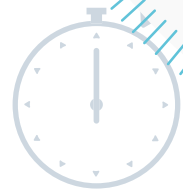

1. Edmonds M, Lázaro-Martínez JL, Alfayate-García JM, Martini J, Petit JM, Rayman G, Lobmann R, Uccioli L, Sauvadet A, Bohbot S, Kerihuel JC, Piaggese A. Sucrose octasulfate dressing versus control dressing in patients with neuroischaemic diabetic foot ulcers (Explorer): an international, multicentre, double-blind, randomised, controlled trial. *Lancet Diabetes Endocrinol.* 2018 Mar;6(3):186-196. 2. Münter KC, Meaume S, Augustin M, Senet P, Kérihuel J.C. The reality of routine practice: a pooled data analysis on chronic wounds treated with TLC-NOSF wound dressings. *J Wound Care.* 2017 Feb; 26 (Sup2): S4-S15. Erratum in: *J Wound Care.* 2017 Mar 2; 26(3): 153

*apart from dark necrosis

Holistic protocol of care for leg ulcers, diabetic foot ulcers and pressure ulcers



What format for what wound ?



For all low to highly exuding wounds

UrgoStart[®]plus



Sizes available
6 x 6 cm
10 x 10 cm
15 x 20 cm

UrgoStart[®]plus
Border



Sizes available
8 x 8 cm
10 x 10 cm
13 x 13 cm
15 x 20 cm



For hard to dress locations & for fragile peri-wound skin

UrgoStart[®]plus
Pad



Sizes available
6 x 6 cm
10 x 10 cm
15 x 20 cm

UrgoStart[®]
Contact



Sizes available
5 x 7 cm
10 x 10 cm
15 x 20 cm



For very highly exuding wounds

UrgoStart[®]plus
Pad

+ secondary absorbant dressing



Sizes available
6 x 6 cm
10 x 10 cm
15 x 20 cm

How to apply?



Remove one side of the protective film. Apply micro-adherent side of the dressing to the wound.



Remove the 2nd side of protective film and gently apply the rest of the dressing.



For UrgoStart[®]Plus: secure the dressing with conforming bandage.

Note
Ensure a certain level of humidity. If the wound is dry, use hydrogel.



Remove one side of the protective film, and grab the dressing with tweezers.



To apply it to the wound bed. Here is an example of cross-shape cut.



Adjust secondary dressing to fit the level of exudates of the wound.



UrgoStart[®] Plus Pad can be cut to fit the shape of the wound.



Remove protective film, and apply micro-adherent side of the dressing to the wound.



For very high levels of exudate, use a super-absorbent secondary dressing on top.



Secure dressing with a conforming bandage & adhesive tape.

Note
Use barrier cream if the wound is very exudative to avoid maceration


Cutting tips with




Cutting tips with




Fit to wound bed shape

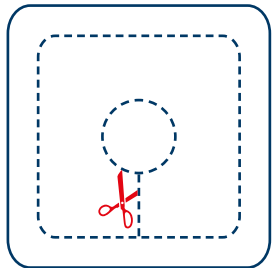


Inter-toes

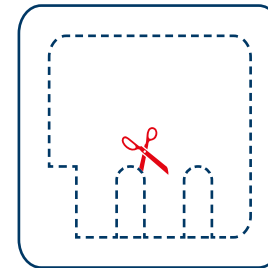



Toes

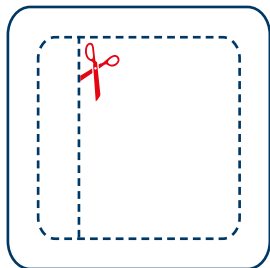

Heels

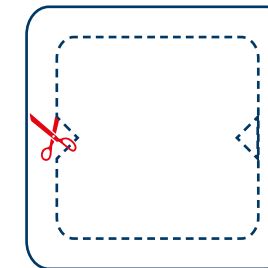

Cavities

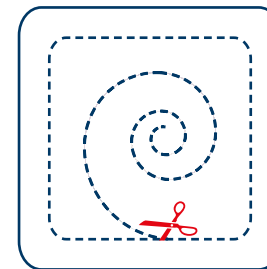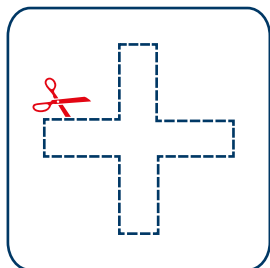

Urgo Medical (Australia) Pty Limited

ABN 29 634 155 829

Level 2, 100 Cubitt Street,
Cremorne, Victoria 3121

Tel +61 (0) 39114 2222

urgomedical.com.au

®/™ all trademarks are the property of their respective owners.

© February 2022 Urgo Medical Australia Pty Ltd

FEB2122V1

